# **ERKA.**

## **Bedienungsanleitung Instruction manual Mode d'emploi Instrucciones de manejo Istruzioni per l'uso**

**Stethoskope  
Stethoscopes  
Stéthoscopes  
Fonendoscopios  
Fonendoscopi**

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**E 3**

Dear customer,

With this stethoscope you have chosen a quality product from ERKA.

We are delighted that you have shown your confidence in ERKA quality and we thank you.

High quality materials perfectly suited to their use give you the guarantee for optimal working.

The stethoscopes are in accordance with the Directive 93/42 EEC dated June 14, 1993 for medical devices.

## **2. Description**

The stethoscope is a unit of chest piece, stethoscope tube and head frame with separate ear tips.

ERKA. Stethoscopes are available with various chest pieces and tubes. Instructions for their use follow.

### *Erkaphon - flat chest piece*

Especially useful for blood pressure measuring. The chest piece will fit under the cuff without problems.

### *Erkaphon - double chest piece*

For the auscultation of high and low frequency tones. Interchangeable.

### *ERKA. Sensitive*

Special equipment: dual membrane and double channel tube. The dual membrane amplifies optimally the pathological sounds in the frequency range of 20-2000 Hz, so that they are clearly audible. The insulated double channel tube prevents transmission fading and disturbing artefacts.

### *ERKA. Precise*

Special equipment: dual membrane, bell and single channel tube. The dual membrane permits auscultation of high and low frequency sounds and especially the bell facilitates perception of lower frequencies. (Cardiology)

## **3. Preparations for use**

For the best auditory quality, please pay attention to the following points:

### *To prevent injury*

The ear pieces should be firmly pressed onto the head frame.

### *Adjustment of the tension of the head frame*

Make sure the frame is adjusted correctly for you.

Modification process:

- **HEAD FRAME IS TOO TIGHT**

To **reduce** the spring tension, firmly grasp each connection of frame/spring between thumb and fore finger and pull apart with downward motion

- **HEAD FRAME IS TOO LOOSE**

To **increase** the spring tension, grasp both connections of frame/spring with one hand and squeeze them together.

This process can be repeated as often as needed.

#### Switching of stethoscopes with double chest piece

Depending on the needed frequency range, select either the membrane or the open bell side. To change hold the slightly bent tube piece in one hand and the chest piece in the other, now turn it 180° until there is a noticeable click.

#### **If the bent tube points to the membrane**

Auscultation with the bell

#### **If the bent tube points to the bell**

Auscultation with the membrane

## **4. Important hints**

- Do not place the stethoscope in extreme heat or cold
- Do not heat-sterilize
- Do not dip in cleaning fluid
- Prevent damage of membrane and tube from sharp and piercing objects
- Do not insert ear pieces too far into ear canal, check for cleanliness regularly and clean as needed
- Never use a stethoscope without ear pieces
- Never use a stethoscope without membrane
- Only use ERKA. original spare parts

## **5. Maintenance and care**

For cleaning we recommend a clean, damp cloth with a mild soap solution. If needed use a **non-aggressive** desinfectant for the membrane. To extend the flexibility of the stethoscope tube we recommend the use of a vinyl protectant.

**For customers with latex allergy  
ERKA. of course carries latexfree products.**

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