Multipad

Sistema de posicionamiento de pacientes de utilización universal para RM y TC
Sistema de posicionamento do doente, universalmente utilizável em imagiologia por RM e TC
Universelt anvendeligt patientpositioneringssystem til MR- og CT-scanning
Yleiskäyttöinen potilaan asemointijärjestelmä MR- ja CT-kuvantamiseen
Universelt pasientposjoneringsssystem for MR- og CT-bilder
Universellt användbart patientpositioneringssystem för magnetresonans- och datortomografi
Universeel toepasbaar patiëntpositionerend systeem ten behoeve van MR- en CT-opnamen
ระบบกำหนดตำแหน่งผู้ป่วยเอกซเรย์สำหรับ MR และการจับภาพ CT
Универсальная система для позиционирования пациента при МРТ и КТ
Uniwersalny system do stabilizacji pacjenta do diagnostyki obrazowej MR i CT
Alhliða skorðunarbúnaður fyrir sjúklinga við segulómmyndun og tölvusneiðmyndun
CT & MRI 펄텍 공기 환자 보조 장치
MR 及 CT 通用病患固定裝置
CT ve CT görüntüleme için evrensel pozisyonlama seti
Universaalne positsioneerimise abivahend MR ja CT uuringutel
Univerzální polohovací pomůcka pro MR a CT vyšetření

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Based on information received from our suppliers, the transparent foil used for the Multipad contains <0.5 wt% of 2-(2H-benzotriazol-2-yl)-4,6-ditertpentylphenol (UV328, CAS: 25973-55-1), which is currently on the Candidate List of SVHCs under the REACH Regulation (EC) No 1907/2006.

Nonetheless, the Multipad fulfills the essential requirements for CE-marking (including biocompatibility) and does not harm patients or users.







Multipad

Instructions for Use
Gebruiksaanweisung
Mode d'emploi
Istruzioni per l'uso
使用説明書
Instrucciones de uso
Instruções de utilização
Brugsanvisning
Käyttöohje
Bruksanvisning
Gebruiksaanwijzing
คู่มือการใช้งาน
Инструкция по эксплуатации
Instrukcja obsługi
Notkunarleiddbeiningar
사용 설명서
使用説明
Kullanım Kitabı
Kasutusjuhend
Návod k použití

Pearl Technology AG
Wiesenstrasse 33
CH – 8952 Schlieren
info@pearl-technology.ch
www.pearl-technology.ch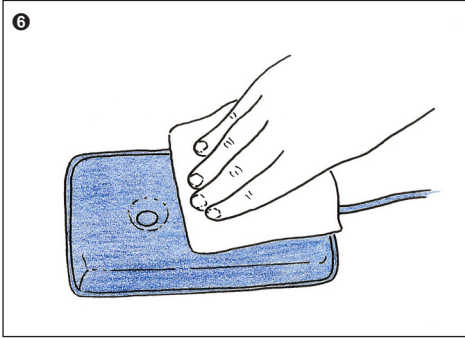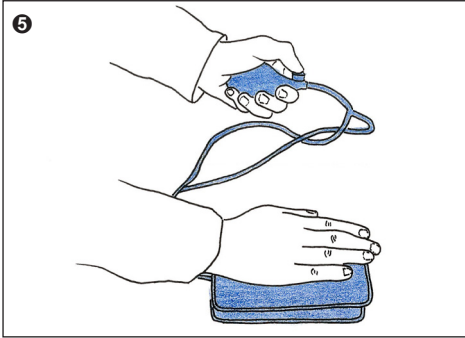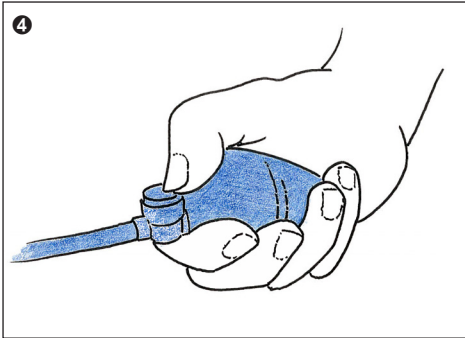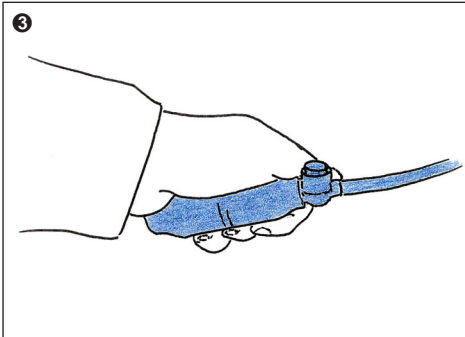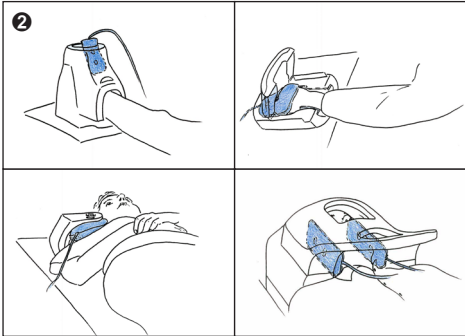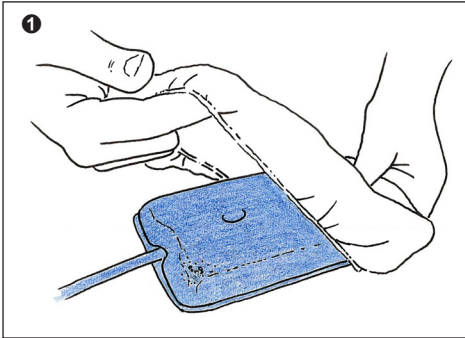

Universal patient positioning aid for MR and CT imaging

- ❶ Before using on a patient, cover the disinfected Multipad with a single-use sanitary cover (Cover Multipad).
Note: Using the cover is essential for sanitary reasons as well as to protect the positioning system.
- ❷ Move the patient into the correct position in preparation for the examination. If necessary, position the Multipad between the part of the body that needs stabilizing and the MR coil or CT support.
Note: The polystyrene pearls should face the patient.
- ❸ Pump up the Multipad until the required degree of support has been attained. Do not over-pump. The patient should find the pressure pleasant. Position the pump and tube to avoid them from getting caught when the patient is moved into the scanner. Position the pump far away from the field of view.
- ❹ Once the examination is complete (see instructions provided by the device manufacturer), deflate the Multipad by pressing the valve on the hand pump. Remove the Multipad from the patient.
- ❺ Dispose of the single-use sanitary cover (Cover Multipad) and squeeze the remaining air out of the Multipad by hand.
- ❻ Clean the Multipad using any readily available hospital disinfectant. The disinfectant should be applied as directed but thoroughly wiped off after application.
Note: Do not immerse the Multipad in cleaning fluid. Optimal functionality guaranteed until red indication line appears on the Multipad.

Contraindications: Do not use the product with children below the age of 3. Do not use the product for purposes other than those described.

MR MR safe

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⚠ Do not sterilize. Do not allow direct contact with the skin. Use single-use sanitary cover. Do not use in case of acute open wounds. Do not inflate with compressed air. Position the pump far away from the field of view. Caution: Federal law restricts this device to sale by or on the order of a licensed health-care professional (Rx only).

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